


# User guide

.bock<sup>///</sup>

## Hand controller to suit bock<sup>®</sup> Practico beds

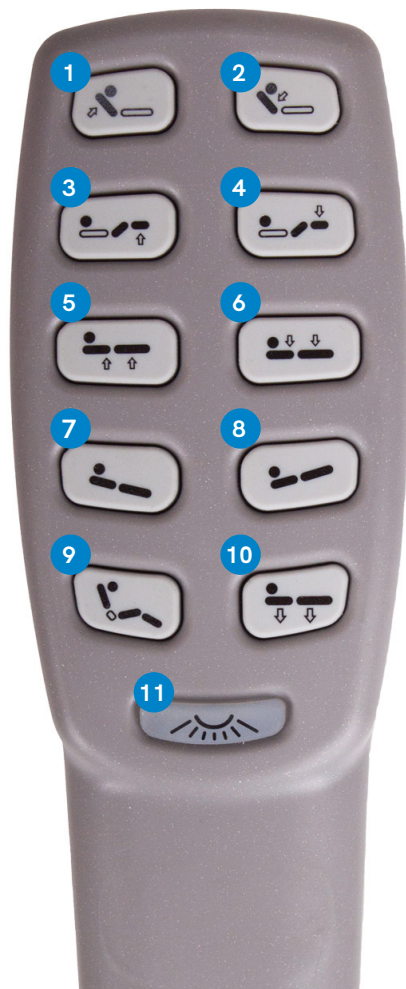

### Hand controller functions

- |  |  |
|--|--|
| 1. Back rest upwards   | 2. Back rest downwards   |
| 3. Knee break upwards  | 4. Knee break downwards  |
| 5. Bed height up   | 6. Bed height down (standard height)                                     |
| 7. Reverse Trendelenburg (tilt whole bed - head up) <a href="#">See note C</a> | 8. Trendelenburg (tilt whole bed - head down) <a href="#">See note C</a> |
| 9. Comfort sitting position (upwards only) <a href="#">To lower see note A</a> | 10. Bed height down (ultra-low height) <a href="#">See note B</a>        |
| 11. Small glow light on hand controller  |  |

**Note A:** To return the bed to a flat lying surface all positions must be lowered separately. Press button 2, 4, and 6. All three buttons can be pressed at the same time.


**Note B:** To enable the ultra-low height the lying surface needs to be flat.

**Note C:** To return bed to flat lying surface press button 6.

### Locking functions


The hand control comes with an integrated disabling function that can be activated and deactivated with the corresponding key. To disable the entire electrical function, insert the key in the key lock on the reverse of the controller and turn the lock function on or off with a corresponding twist of the key.

- A.** Key for locking device

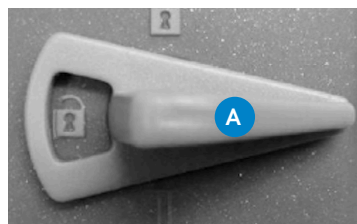

For more information please refer to full user guide.

### Locking device

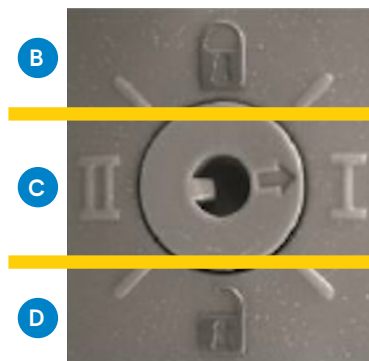

- B.** All functions locked
- C.** All functions working
- D.** Ultra low and reverse Trendelenburg (head down) locked, all other functions working

### Limited warranty

1. This warranty applies only to the original purchaser of CUBRO LTD's product (who must be a purchaser who is resident or carrying on business in New Zealand and who has purchased the product directly from CUBRO LTD, or from a CUBRO LTD authorised distributor or reseller in New Zealand) and it is not transferable to any other person or entity.
2. This warranty replaces all conditions, warranties or guarantees that might be implied by law in favour of any person, which are excluded to the fullest extent permissible by law. Where the Consumer Guarantees Act would apply but the purchaser acquires or holds themselves out as acquiring any product for business purposes, the guarantees provided under the Consumer Guarantees Act are excluded in relation to that product.
3. CUBRO LTD warrants the product to be free from defects in materials and workmanship from date of purchase for a period of two years (24 months). This warranty does not cover any damage, defect, expense or loss of any kind caused by accident, misuse, abuse, neglect, negligence, alteration or modification (which includes the use of unauthorised parts or attachments), improper service, repair by other than authorised personnel or any defects not related to materials or workmanship. Wear of components in normal operation and failures resulting therefrom are excluded from this warranty.
4. CUBRO LTD has no obligation to the purchaser and is not obligated to honour all or any part of this warranty unless the following procedure is followed by the purchaser:
  - a. Before making a warranty claim, the purchaser should ensure that the product is defective by following standard 'trouble shooting' procedures and be able to attribute the fault to a defect in materials and/or workmanship of CUBRO LTD;
  - b. If CUBRO LTD determines that the product is not defective, or that the warranty claim is otherwise invalid, CUBRO LTD shall charge the purchaser a handling and servicing fee as well as any freights costs.
  - c. Upon receiving notice from the purchaser of an alleged defect in a product (which defect should be reported to CUBRO LTD immediately), CUBRO LTD will issue a serialised return management authorisation (RMA). The purchaser must then return the entire unit or remove, at the purchaser's cost, the defective component part(s) identified, pack the unit or the component part(s) in a manner to avoid shipping damage and to ship the unit or the component part(s) to either CUBRO LTD, or a service centre as specified by CUBRO LTD, within 30 days of the date of the serialized return authorisation date.
  - d. If CUBRO LTD require additional information relating to the use of the product by the purchaser or any other relevant information, the purchaser will supply such information as soon as practically possible and in such a manner as CUBRO LTD may reasonably require.
  - e. If CUBRO LTD access the purchaser's claim, it will either repair or replace the product, or repair or replace the defect in workmanship, as it may determine in its sole discretion.
5. Proper selection of a specific product for a specific application and operating environment, and its compatibility with other equipment is the purchaser's responsibility. CUBRO LTD does not warrant the performance of its products or their suitability for a particular purpose.
6. CUBRO LTD shall not be liable under any heading (whether in contract, negligence, or otherwise) for any consequential, indirect or incidental loss or damages of any kind and the maximum aggregate liability of CUBRO LTD (under any one or more headings) shall be the purchase price of the product.

© Cubro Limited 2021

*ALL RIGHTS RESERVED: No part of this publication shall be adapted, modified, reproduced, copied, or transmitted in any form or by any means including written electronic, mechanical, reprographic, photocopying, or recording means. Furthermore, this publication shall not be stored in whole, part, adapted, or modified form, in or for any retrieval system of any nature without the written permission of the copyright holder. Applications for authorisation of a reserved right of copyright owner shall be made in writing to the publisher. WARNING: The doing of any unauthorised act in relation to a copyright work may result in both a civil claim for damages and criminal prosecution.*